

REUSABLE ECO EAR CLEANER INSTRUCTION MANUAL



These reusable ear wax cleaners have been a real everyday helper for gentle ear care at home. Designed with soft silicone spiral tips, they offer a more controlled and considered alternative to cotton buds, which can push wax further into the ear.

The reusable tips are easy to wash and reuse, helping reduce single-use plastic waste. One handle can be shared across a household, with each person using their own silicone tip, making it both practical and economical for families.

Simple, reusable, and designed to be used gently, this set supports regular ear hygiene with care and intention.

Great for:

- Gentle ear care at home
- Reducing single-use cotton buds
- Shared family use with individual tips
- Easy cleaning and reuse
- People wanting a more controlled alternative to cotton buds

Features:

- Soft silicone spiral tips designed for gentle use
- Durable handle for controlled grip
- Reusable and easy to clean
- One handle with multiple individual tips
- Helps reduce disposable ear-cleaning waste

Image Description: A reusable ear wax cleaning tool with a blue-and-white handle and soft silicone spiral tips. Images show the tool, interchangeable tips, and a person carefully using the device near their ear. Multiple tips are included for individual use.

Details:

Material: PP durable hard plastic with soft silicone tip

Size: 10cm handle length (12.1cm with tip attached), 2cm diameter, 2.7cm length for just the tip itself.

Weight: 27 grams

Package includes: 1x handle + 16 reusable tips

Age Guidance: Recommended for ages 8+. Adult supervision required for children. Not a toy.



r

How to Use:

1. Choose one silicone tip and store the others for future use
2. Align the grooves and click the tip securely onto the handle
3. Gently insert the silicone tip into the outer ear canal
4. Slowly rotate in the direction indicated on the handle while withdrawing
5. Rinse the tip under warm water with mild soap to remove wax
6. Wash and dry the tool after use, then store safely
7. Use gently and repeat as needed. Do not force.

Care Instructions:

- Wash tips with warm water and mild soap after each use
- Allow to fully air dry before storing
- Store tips separately if sharing a handle
- Do not sterilise with heat or boiling water

Medical Tips / Safety Notes:

- Use gently and only in the outer ear canal
- Do not insert deeply or force the tool
- Stop use immediately if pain, dizziness, or discomfort occurs
- Do not use if you have ear pain, infection, a perforated eardrum, recent ear surgery, or ear tubes in place
- If wax build-up persists or hearing is affected, consult a GP, audiologist, or healthcare professional
- Always supervise use with children